

## GENERAL INFORMATION FOR SAMPLE SUBMISSION



### SHIPPING FROM EU MEMBER STATES

#### Submitting original/native samples (e. g. swabs, serum, tissue, feces etc.)

Samples must be accompanied by a commercial document and by our submission form. The CN-code on the commercial document can be indicated as follows:

- **CN-code for isolates:** 30024900
- **CN-code for blood samples:** 30029030
- **CN-code for other diagnostic material:** 05119985

#### Submitting FTA-cards/AniCards

Cards should be accompanied by our submission form.

SUBMISSION FORM INCL. COMMERCIAL DOCUMENT  
AND PACKAGING INSTRUCTIONS



[www.anicon.eu/swine-downloads](http://www.anicon.eu/swine-downloads)



### PACKAGING

Samples sent to our laboratory should only contain Category B material acc. to UN3373 which includes – among others – animal material that gets transported for the purpose of diagnosis or examination. Such material includes excreta and blood as well as fluids and other tissues.

Original/native samples belonging to the Category B should be packed according to the packaging instruction P650, which implies to UN 3373. The package should consist of three components:

- a) a primary leakage proof packaging
- b) a secondary packaging with sufficient absorbing material
- c) an outer packing

FOR FURTHER INFORMATION SEE THE PACKAGING  
INSTRUCTIONS ON OUR WEBSITE



[www.anicon.eu/poultry-downloads](http://www.anicon.eu/poultry-downloads)



### SHIPPING FROM NON-EU MEMBER STATES

#### Submitting original/native samples (e. g. swabs, serum, tissue, feces etc.)

Sending original samples from non-EU-countries is restricted. Please contact us in any case for possibilities and further information.

#### Submitting FTA-cards/AniCards

Cards should be accompanied by our submission form.

For the shipment of FTA-cards/AniCards from non-EU member states, please indicate the following HS-code and EORI-number to the shipping-company:

HS-code: 48119000

EORI-number: DE6173438